

Dolphin Neurostim Kit

The Dolphin Neurostim Kit Includes:

- ◆ The portable Dolphin Neurostim device
- ◆ Easy-to-use instruction manual
- ◆ Toll-free support line
- ◆ Portable carrying case
- ◆ Extension probe (optional accessory)



Electrical Specifications

Power Source: 9 Volt Battery

Wave Formation: Square Direct Current (DC wave)

Voltage Output: 7 - 9 Volts

Current Output: 50 - 800 microamps (amplitude is inversely proportional to skin resistance $V = I \times R$)

During the "search" mode: testing from 0 to 100 kilohms gives the following parameters: voltage 0.15 - 4V, width 0.3 - 3.6 millisecc and frequency between 200Hz and 2.0KHz.

During stimulation phase, parameters are: 0 to 8V with a frequency of 2.5Hz ($\pm 10\%$). Pulses in this mode also vary with the intensity setting. When set at maximum intensity the current is 60 microamps for 100 kilohms and 400 microamps for 10 kilohms.

NOTE: Current is inversely proportional to skin resistance and intensity will naturally vary depending on skin resistance of the patients' Acupuncture points. The lower the resistance of the Acupuncture point the stronger the stimulation felt by the patient. Tenderness of the points will vary with severity of the condition. Maximum peak pulse width is 200 milliseconds for the + (positive) stimulation mode and 230 milliseconds for the - (negative) stimulation mode. The graphs below are representative of the Dolphin MPS unit output in the positive and bipolar stimulation modes.



Certifications and Approvals

The Dolphin Neurostim is approved for sale by **Health Canada** and the **FDA**. We are also **ISO 13485** certified and have **CE Certification** for Europe.

www.DolphinNeurostim.com

Toll-Free: 1-800-567-PAIN (7246)

Phone: 416-253-6060

Fax: 416-253-1911

info@dolphinneurostim.com

© **Acumed**

Dolphin Neurostim

All Natural, Drug-Free Pain Relief



Eliminate Chronic Pain

Health Science & Electronic Technology Working Together

The Dolphin Neurostim electrically locates and stimulates specific treatment points (motor, trigger, and acupuncture), which have been proven to help relieve many chronic pain syndromes.

Applying the Dolphin Neurostim to tightened or contracted muscles, or to treatment points relating to them, seems to “release” the muscles and permits increased reinnervation of their neural pathways. This process allows the patient’s pain levels to be substantially decreased.

Bill Bowling, DPT

What Does it Do?

Relaxes muscles
Reduces nervous tension
Releases fascia
Releases scars
Increases range of motion
Improves circulation
Duplicates acupuncture
All in one simple to use modality!

What Can it Help?

Arthritic Pain
Head Pain
Back Pain
Sciatica

Knee Pain
Fibromyalgia
Inflamed Joints
Carpal Tunnel

Bursitis
Sports Injuries
Tendonitis
And more!

*Take control
of pain and manage
it safely*



*Quickly
relax tightened
muscles, immediately
improve range
of motion and
circulation*

Q&A

Is the Dolphin Neurostim difficult to use?

We think you'll find that both our self-help unit and our systematic approach to pain management are extremely user-friendly. To shorten your learning curve, we include an easy-to-follow treatment manual with every Dolphin Neurostim. Additionally, we provide the best toll-free support line in the industry.

How effective is the Dolphin Neurostim for pain relief?

The majority of our patients, therapists and doctors have reported 50-75% relief within one or two treatments. Properly applied, almost every patient will respond favourably within 2-3 applications!

What does the medical profession say about the Dolphin Neurostim?

Practitioners all over the world are discovering the benefits of the Dolphin Neurostim's pain relieving abilities. Our unit is helping thousands of patients in hospitals, chiropractic, massage, and physiotherapy clinics.

Will my insurance company pay for the Dolphin Neurostim?

Most private health plans, WISB, and the DVA will cover TENS devices. In most cases, all that is needed is a Medical Doctor's recommendation.

For more information, please visit www.DolphinNeurostim.com



*Enjoy
natural pain
relief without
adverse side
effects*

Our Guarantee

We are **so confident** you will find great pain relief with your Dolphin Neurostim unit that it comes with our **unconditional 30 day money back guarantee**. If you are not completely satisfied, return the entire system in good condition and we'll refund the purchase price with **no questions asked**.

Testimonials

“The Dolphin Neurostim has revolutionized the way we treat patients with pain. While many treatment approaches have been proposed for patients in chronic pain, it has been the most successful. This method allows to treat not just the areas that hurt, but also the areas that are causing the problem.”

Dr. Chris Centeno, MD

“As a PT treating neuro special needs pediatrics for over 20 years, I have seen results with the Dolphin Neurostim when all other modalities failed. It gives new hope and the parents are always excited about the progress they see. I highly recommend the Dolphin Neurostim for any therapist.”

Julietta Planchart, PT

“As a PTA practicing for over 20 years have never observed any modality as successful as the Dolphin Neurostim for treating chronic pain. The outcomes obtained are truly amazing.”

R. Feldhaus, PTA